

# SIEMENS MAGNETOM TrioTim syngo MR B17

\\USER\Nage\NK-Study\Nigg-Fair Studies 10.10\T1Anatomical-1

TA: 9:14    PAT: Off    Voxel size: 1.0x1.0x1.1 mm    Rel. SNR: 1.00    SIEMENS: tfl

## Properties

Prio Recon	Off
Before measurement	
After measurement	
Load to viewer	On
Inline movie	Off
Auto store images	On
Load to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Start measurement without further preparation	On
Wait for user to start	On
Start measurements	single

## Routine

Slab group 1	
Slabs	1
Dist. factor	50 %
Position	L1.2 A34.7 F18.1
Orientation	Sagittal
Phase enc. dir.	A >> P
Rotation	0.00 deg
Phase oversampling	0 %
Slice oversampling	10.0 %
Slices per slab	160
FoV read	256 mm
FoV phase	93.8 %
Slice thickness	1.10 mm
TR	2300 ms
TE	3.58 ms
Averages	1
Concatenations	1
Filter	Prescan Normalize
Coil elements	HEA;HEP

## Contrast

Magn. preparation	Non-sel. IR
T1	900 ms
Flip angle	10 deg
Fat suppr.	None
Water suppr.	None
Averaging mode	Short term
Reconstruction	Magnitude
Measurements	1
Multiple series	Each measurement

## Resolution

Base resolution	256
Phase resolution	100 %
Slice resolution	100 %
Phase partial Fourier	Off
Slice partial Fourier	Off
Interpolation	Off
PAT mode	None
Matrix Coil Mode	Auto (CP)
Image Filter	Off
Distortion Corr.	Off
Unfiltered images	Off
Prescan Normalize	On
Normalize	Off
B1 filter	Off

Raw filter	Off
Elliptical filter	Off

## Geometry

Multi-slice mode	Single shot
Series	Interleaved

## System

Body	Off
HEP	On
HEA	On
SP4	Off
SP2	Off
SP8	Off
SP6	Off
SP3	Off
SP1	Off
SP7	Off
SP5	Off

Positioning mode	FIX
Table position	H
Table position	0 mm
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Save uncombined	Off
Coil Combine Mode	Sum of Squares
AutoAlign	---
Auto Coil Select	Default

Shim mode	Standard
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Silicone	Off
? Ref. amplitude 1H	0.000 V
Adjustment Tolerance	Auto
Adjust volume	
Position	L1.2 A34.7 F18.1
Orientation	Sagittal
Rotation	0.00 deg
F >> H	256 mm
A >> P	240 mm
R >> L	176 mm

## Physio

1st Signal/Mode	None
Dark blood	Off
Resp. control	Off

## Inline

Subtract	Off
Std-Dev-Sag	Off
Std-Dev-Cor	Off
Std-Dev-Tra	Off
Std-Dev-Time	Off
MIP-Sag	Off
MIP-Cor	Off
MIP-Tra	Off
MIP-Time	Off
Save original images	On

## Sequence

Introduction	On
Dimension	3D

# SIEMENS MAGNETOM TrioTim syngo MR B17

Elliptical scanning	Off
Asymmetric echo	Off
Bandwidth	180 Hz/Px
Flow comp.	No
Echo spacing	8.2 ms
-----	
RF pulse type	Fast
Gradient mode	Fast*
Excitation	Non-sel.
RF spoiling	On