

## SIEMENS MAGNETOM Skyra syngo MR D13

\\USER\Assaf\Autism/ schizophrenia(126265)\ASD\SZ.2013 day 1\cmrrBOLD rest  
TA:7:37 PAT:Off Voxel size:3.0×3.0×3.0 mm Rel. SNR:1.00 :epfid

### Properties

Prio Recon	Off
Load to viewer	On
Inline movie	Off
Auto store images	On
Load to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Wait for user to start	On
Start measurements	single

### Routine

Nr. of slice groups	1
Slices	48
Dist. factor	0 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
AutoAlign	---
Phase oversampling	0 %
FoV read	240 mm
FoV phase	100.0 %
Slice thickness	3.00 mm
TR	475 ms
TE	30.00 ms
Multi-band accel. factor	8
Filter	Prescan Normalize
Coil elements	HEA;HEP

### Contrast

MTC	Off
Magn. preparation	None
Flip angle	60 deg
Fat suppr.	Fat sat.
Averaging mode	Long term
Measurements	947
Delay in TR	0 ms
Reconstruction	Magnitude
Multiple series	Off

## Resolution

Base resolution	80
Phase resolution	100 %
Phase partial Fourier	Off
Interpolation	Off
PAT mode	None
Distortion Corr.	Off
Hamming	Off
Prescan Normalize	On
Raw filter	Off
Elliptical filter	Off

## Geometry

Nr. of slice groups	1
Slices	48
Dist. factor	0 %
Position	Isocenter
Phase enc. dir.	A >> P
Phase oversampling	0 %
Multi-slice mode	Interleaved
Series	Interleaved
Nr. of sat. regions	0
Position mode	L-P-H
Fat suppr.	Fat sat.
Special sat.	None
Special sat.	None
Table position	P

## System

Body	Off
HEP	On
HEA	On
SP5	Off
SP6	Off
SP7	Off
SP8	Off
SP1	Off
SP2	Off
SP3	Off
SP4	Off
Position mode	L-P-H
Positioning mode	REF
Table position	H
Table position	0 mm
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Sum of Squares
AutoAlign	---
Coil Select Mode	Default
Shim mode	Standard
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto
? Ref. amplitude 1H	0.000 V
Position	Isocenter
Rotation	0.00 deg
R >> L	240 mm
A >> P	240 mm
F >> H	144 mm
Frequency 1H	123.246551 MHz
Correction factor	1
MBExc 1H	546.973 V
Gain	High
Table position	0 mm
Img. Scale. Cor.	1.000

## Physio

1st Signal/Mode	None
Magn. preparation	None

## Inline

Distortion correction	Off
-----------------------	-----

## Sequence

Introduction	Off
Averaging mode	Long term
Multi-slice mode	Interleaved
Bandwidth	2604 Hz/Px
Flow comp.	No
Free echo spacing	Off
Echo spacing	0.55 ms
EPI factor	80
Gradient mode	Fast
RF spoiling	Off
Online multi-band recon.	Online
Physio recording	Off
Triggering scheme	Standard
TX/RX delta frequency	0 Hz
TX Nucleus	None
TX delta frequency	0 Hz
Coil elements	HEA;HEP
Acquisition duration	0 ms

## BOLD

GLM Statistics	Off
Dynamic t-maps	Off
Ignore meas. at start	0
Ignore after transition	0
Model transition states	On
Temp. highpass filter	On
Threshold	4.00
Paradigm size	20
Motion correction	Off
Spatial filter	Off
Delay in TR	0 ms
Distortion Corr.	Off