

PATIENT POSITION

Patient Entry	Head First
Patient Position	Supine
Coil Configuration	0
Plane	SAGITTAL
Series Description	3D_T1_Sag- Structural_(FSPGR_BRAVO)

SCAN TIMING

Flip Angle	12
TI	650
Receiver Bandwidth	25.00

IMAGE ENHANCE

Filter Choice	None
---------------	------

GATING/TRIGGER

Auto Trigger Type	Off
-------------------	-----

MULTI-PHASE

Seperate Series	0
Mask Phase	0
Mask Pause	0

TRICKS

Pause On/Off	On
Auto Subtract	0
Auto SCIC	Off

IMAGING PARAMETERS

Imaging Mode	3D
Pulse Sequence	BRAVO
Imaging Options	EDR, Fast, IrP, ARC

SCAN RANGE

FOV	25.6
Slice Thickness	1.0
Location per Slab	220
Overlap Locations	0

ACQ TIMING

Freq	256
Phase	256
Freq DIR	S/I
NEX	1.00
Phase FOV	1.00
Auto Shim	Auto
Phase Correction	No

FMRI

PSD Trigger	Internal
Slice Order	Interleaved
View Order	Bottom/Up
# of Repetitions REST	0
# of Repetitions ACTIVE	0

DIFFUSION

Recon All Images	On
------------------	----

CONTRAST

Contrast Yes/No	No
-----------------	----