

SIEMENS MAGNETOM TrioTim syngo MR B13

\\USER\DICKSTEIN\PEDIMIND\ROUTINE\ep2d 256

TA: 8:36 PAT: Off Voxel size: 3.0x3.0x3.0 mm Rel. SNR: 1.00 SIEMENS: ep2d_pace

Properties

Prio Recon	Off
Voice output before	
Voice output after	
Load to viewer	On
Inline movie	Off
Auto store images	On
Load to stamp segments	Off
Load to graphic segments	Off
Auto open inline display	Off
AutoAlign Spine	Off
Start measurement without further preparation	On
Wait for user to start	On
Start measurements	single

Routine

Slice group 1	
Slices	35
Dist. factor	0 %
Position	R0.7 A34.9 H23.5
Orientation	Transversal
Phase enc. dir.	A >> P
Rotation	0.00 deg
Phase oversampling	0 %
FoV read	192 mm
FoV phase	100.0 %
Slice thickness	3.0 mm
TR	2000 ms
TE	25 ms
Averages	1
Concatenations	1
Filter	None
Coil elements	HEA;HEP

Contrast

MTC	Off
Flip angle	90 deg
Fat suppr.	Fat sat.
Averaging mode	Long term
Reconstruction	Magnitude
Measurements	256
Delay in TR	0 ms
Multiple series	Off

Resolution

Base resolution	64
Phase resolution	100 %
Phase partial Fourier	Off
Filter 1	
Raw filter	Off
Filter 2	
Distortion Corr.	Off
Filter 3	
Prescan Normalize	Off
Filter 4	
Elliptical filter	Off
Filter 5	
Hamming	Off
Trajectory	Cartesian
Interpolation	Off
PAT mode	None
Matrix Coil Mode	Auto (CP)

Geometry

Multi-slice mode	Interleaved
Series	Interleaved
Special sat.	None

System

Body	Off
HEP	On
HEA	On
SP4	Off
SP2	Off
SP8	Off
SP6	Off
SP3	Off
SP1	Off
SP7	Off
SP5	Off

Positioning mode	REF
Table position	H
Table position	0 mm
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Adaptive Combine

Shim mode	Standard
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Silicone	Off
Ref. amplitude 1H	288.591 V
Adjustment Tolerance	Auto
Adjust volume	
Position	R0.7 A34.9 H23.5
Orientation	Transversal
Rotation	0.00 deg
R >> L	192 mm
A >> P	192 mm
F >> H	105 mm

Physio

1st Signal/Mode	None
-----------------	------

BOLD

Threshold	4.00
Window	Growing
Dynamic t-maps	On
Starting ignore meas	0
Paradigm size	4
Meas[1]	Baseline
Meas[2]	Baseline
Meas[3]	Baseline
Meas[4]	Baseline
Motion correction	Off
Spatial filter	Off

Sequence

Introduction	Off
Bandwidth	3126 Hz/Px
Free echo spacing	On
Echo spacing	0.51 ms
EPI factor	64
RF pulse type	Normal

SIEMENS MAGNETOM TrioTim syngo MR B13

Gradient mode
Flip angle mode

Fast*
Constant