

SIEMENS MAGNETOM TrioTim syngo MR B13

\\USER\head\FRAN\BOLD\Mprage

TA: 8:09

PAT: 2

Voxel size: 1.0×1.0×1.0 mm

Rel. SNR: 1.00

SIEMENS: tfl

Properties

Prio Recon	Off
Voice output before	
Voice output after	
Load to viewer	On
Inline movie	Off
Auto store images	On
Load to stamp segments	Off
Load to graphic segments	Off
Auto open inline display	Off
AutoAlign Spine	Off
Start measurement without further preparation	On
Wait for user to start	On
Start measurements	single

Elliptical filter	Off
Filter 6	
Image Filter	Off
Interpolation	Off
PAT mode	GRAPPA
Accel. factor PE	2
Ref. lines PE	24
Accel. factor 3D	1
Matrix Coil Mode	Triple
Reference scan mode	Integrated

Geometry

Multi-slice mode	Single shot
Series	Ascending

Routine

Slab group 1	
Slabs	1
Dist. factor	50 %
Position	R3.0 A10.0 H0.0
Orientation	Sagittal
Phase enc. dir.	A >> P
Rotation	14.50 deg
Phase oversampling	50 %
Slice oversampling	0 %
Slices per slab	256
FoV read	256 mm
FoV phase	100.0 %
Slice thickness	1.00 mm
TR	2400 ms
TE	3.08 ms
Averages	1
Concatenations	1
Filter	Prescan Normalize
Coil elements	HEA;HEP

System

Body	Off
HEP	On
HEA	On
SP4	Off
SP2	Off
SP8	Off
SP6	Off
SP3	Off
SP1	Off
SP7	Off
SP5	Off
Positioning mode	REF
Table position	H
Table position	0 mm
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Save uncombined	Off
Coil Combine Mode	Adaptive Combine

Contrast

Magn. preparation	Non-sel. IR
TI	1000 ms
Flip angle	8 deg
Fat suppr.	None
Water suppr.	None
Averaging mode	Long term
Reconstruction	Magnitude
Measurements	1
Multiple series	Off

Shim mode	Standard
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Silicone	Off
Ref. amplitude 1H	237.849 V
Adjustment Tolerance	Auto
Adjust volume	
Position	R3.0 A10.0 H0.0
Orientation	Sagittal
Rotation	14.50 deg
F >> H	256 mm
A >> P	256 mm
R >> L	256 mm

Resolution

Base resolution	256
Phase resolution	100 %
Slice resolution	100 %
Phase partial Fourier	Off
Slice partial Fourier	Off
Filter 1	
Raw filter	Off
Filter 2	
Distortion Corr.	Off
Filter 3	
Prescan Normalize	On
Unfiltered images	Off
Filter 4	
Normalize	Off
Filter 5	

Physio

1st Signal/Mode	None
Dark blood	Off
Resp. control	Off

Inline

Subtract	Off
Std-Dev-Sag	Off
Std-Dev-Cor	Off
Std-Dev-Tra	Off
Std-Dev-Time	Off
MIP-Sag	Off

SIEMENS MAGNETOM TrioTim syngo MR B13

MIP-Cor	Off
MIP-Tra	Off
MIP-Time	Off
Save original images	On

Sequence

Introduction	On
Dimension	3D
Elliptical scanning	Off
Asymmetric echo	Off
Bandwidth	220 Hz/Px
Flow comp.	No
Echo spacing	7.2 ms
RF pulse type	Fast
Gradient mode	Fast*
Excitation	Non-sel.
RF spoiling	On