

fMRI RestState

PATIENT POSITION	
Patient Entry	Head First
Patient Position	Supine
Coil Configuration	32 Ch Head Coil
Plane	AXIAL
Series Description	fMRI RestState
SCAN TIMING	
Flip Angle	90
TE	40.0
Number of Echoes	1
TR	2000.0
Number of Shots	1
IMAGE ENHANCE	
Filter Choice	None
GATING/TRIGGER	
Auto Trigger Type	Off
Auto Trigger Window	Off
FMRI	
Brain Wave Real Time	1
Paradigm String	RestingState Hipnosis 180
Paradigm UID	1.2.840.113819.3.116520967544.1317943857.627
Initial State	Control
PSD Trigger	Internal
Slice Order	Interleaved
View Order	Bottom/Up
# of Repetitions REST	6
# of Repetitions ACTIVE	6
# of Dummy Acquisition	5
SAT	
Tag Type	None
TRICKS	
Pause On/Off	On
Auto Subtract	0
Auto SCIC	Off

fMRI RestState

IMAGING PARAMETERS	
Imaging Mode	2D
Pulse Sequence	Gradient Echo
Imaging Options	EPI, Asset, FMRI
n/a	9
SCAN RANGE	
FOV	25.6
Slice Thickness	4.0
Slice Spacing	0.0
GRXOPT	0
Start Location 1	I31.4
End Location 1	S104.6
End Location 2	R5.8
End Location 3	A33.3
Center of Location Start	R5.8
Center of Location End	A33.3
Number of Slices	35
Slice for 3 Plane Localizaion	0
Slice for 3 Plane Localizaion	0
Slice for 3 Plane Localizaion	0
Slice for 3 Plane Localizaion	0
Space per Plane 1	0.0
Space per Plane 2	0.0
Table Delta	0.00
ACQ TIMING	
Freq	64
Phase	64
Freq DIR	R/L
NEX	1.00
Phase FOV	1.00
Auto Shim	Auto
Phase Correction	Yes
USER CVS	
User CV0	1.00
In-range AutoTR Control	0
User CV Mask	1
MULTI-PHASE	
Slice per Location	180
Delay after Acquisition	0
Seperate Series	0
Delay after Acquisition without AV	0
Mask Phase	0
Mask Pause	0
DIFFUSION	
Recon All Images	On
CONTRAST	
Contrast Yes/No	No
Contrast Amount	Yes

fMRI RestState