

## PHILIPS Intera Achieva 1.5 T

SAG T1 3D	MPRAGE	Voxel 1.00x1.00x1.00	Rel. SNR 0.98
TE 5.6 TR 25	Uniformity CLEAR	AP (mm) 240	RL (mm) 170

<table style="width: 100%; border-collapse: collapse;"> <tr><td>Total scan duration</td><td>04 :37.9</td></tr> <tr><td>Rel. SNR</td><td>0.978</td></tr> <tr><td>Act. TR/TE (ms)</td><td>25/5.6</td></tr> <tr><td>ACQ matrix M x P</td><td>240 x 240</td></tr> <tr><td>ACQ voxel MPS (mm)</td><td>1.00 / 1.00 / 1.00</td></tr> <tr><td>REC voxel MPS (mm)</td><td>0.94 / 0.94 / 1.00</td></tr> <tr><td>Scan percentage (%)</td><td>100</td></tr> <tr><td>Act. WFS (pix) / BW (Hz)</td><td>1533 / 141.7</td></tr> <tr><td>Min. WFS (pix) / Max BV (Hz)</td><td>0.425 / 510.6</td></tr> <tr><td>Min. TR/TE (ms)</td><td>12 / 4.7</td></tr> <tr><td>SAR / local torso</td><td>&lt; 8%</td></tr> <tr><td>Whole body / level</td><td>&lt; 0.2 W/Kg/normal</td></tr> <tr><td>SED</td><td>&lt; 0.1 Kj/Kg</td></tr> <tr><td>B1+rms</td><td>119 uT</td></tr> <tr><td>Max B1+rms</td><td>119 uT</td></tr> <tr><td>PNS / level</td><td>22% / normal</td></tr> <tr><td>dB/dt</td><td>12.8 T/s</td></tr> <tr><td>Sound Pressure Level (dB)</td><td>-6.6</td></tr> </table>	Total scan duration	04 :37.9	Rel. SNR	0.978	Act. TR/TE (ms)	25/5.6	ACQ matrix M x P	240 x 240	ACQ voxel MPS (mm)	1.00 / 1.00 / 1.00	REC voxel MPS (mm)	0.94 / 0.94 / 1.00	Scan percentage (%)	100	Act. WFS (pix) / BW (Hz)	1533 / 141.7	Min. WFS (pix) / Max BV (Hz)	0.425 / 510.6	Min. TR/TE (ms)	12 / 4.7	SAR / local torso	< 8%	Whole body / level	< 0.2 W/Kg/normal	SED	< 0.1 Kj/Kg	B1+rms	119 uT	Max B1+rms	119 uT	PNS / level	22% / normal	dB/dt	12.8 T/s	Sound Pressure Level (dB)	-6.6	<table style="width: 100%; border-collapse: collapse;"> <tr><td>PlanAlign</td><td>no</td></tr> <tr><td>REST slab</td><td>0</td></tr> <tr><td>Interactive positioning</td><td>no</td></tr> <tr><td colspan="2"><b>Contrast</b></td></tr> <tr><td>Scan type</td><td>Imaging</td></tr> <tr><td>Scan mode</td><td>3D</td></tr> <tr><td>    technique</td><td>FFE</td></tr> <tr><td>Contrast enhancement</td><td>T1</td></tr> <tr><td>Acquisition mode</td><td>cartesian</td></tr> <tr><td>Fast Imaging mode</td><td>none</td></tr> <tr><td>Echoes</td><td>1</td></tr> <tr><td>    Partial echo</td><td>no</td></tr> <tr><td>    Shifted echo</td><td>no</td></tr> <tr><td>TE</td><td>user defined</td></tr> <tr><td>    (ms)</td><td>5.6</td></tr> <tr><td>Flip angle (deg)</td><td>30</td></tr> <tr><td>TR</td><td>user defined</td></tr> <tr><td>    (ms)</td><td>25</td></tr> <tr><td>Halfscan</td><td>no</td></tr> <tr><td>Water-fat shift</td><td>maximum</td></tr> <tr><td>Shim</td><td>default</td></tr> <tr><td>mDIXON</td><td>no</td></tr> <tr><td>Fat suppression</td><td>no</td></tr> <tr><td>Water suppression</td><td>no</td></tr> <tr><td>MTC</td><td>no</td></tr> <tr><td>Diffusion mode</td><td>no</td></tr> <tr><td>SAR mode</td><td>high</td></tr> <tr><td>B1 mode</td><td>default</td></tr> <tr><td>PNS mode</td><td>low</td></tr> <tr><td>Gradient mode</td><td>default</td></tr> <tr><td>SoftTone mode</td><td>no</td></tr> </table>	PlanAlign	no	REST slab	0	Interactive positioning	no	<b>Contrast</b>		Scan type	Imaging	Scan mode	3D	technique	FFE	Contrast enhancement	T1	Acquisition mode	cartesian	Fast Imaging mode	none	Echoes	1	Partial echo	no	Shifted echo	no	TE	user defined	(ms)	5.6	Flip angle (deg)	30	TR	user defined	(ms)	25	Halfscan	no	Water-fat shift	maximum	Shim	default	mDIXON	no	Fat suppression	no	Water suppression	no	MTC	no	Diffusion mode	no	SAR mode	high	B1 mode	default	PNS mode	low	Gradient mode	default	SoftTone mode	no
Total scan duration	04 :37.9																																																																																																		
Rel. SNR	0.978																																																																																																		
Act. TR/TE (ms)	25/5.6																																																																																																		
ACQ matrix M x P	240 x 240																																																																																																		
ACQ voxel MPS (mm)	1.00 / 1.00 / 1.00																																																																																																		
REC voxel MPS (mm)	0.94 / 0.94 / 1.00																																																																																																		
Scan percentage (%)	100																																																																																																		
Act. WFS (pix) / BW (Hz)	1533 / 141.7																																																																																																		
Min. WFS (pix) / Max BV (Hz)	0.425 / 510.6																																																																																																		
Min. TR/TE (ms)	12 / 4.7																																																																																																		
SAR / local torso	< 8%																																																																																																		
Whole body / level	< 0.2 W/Kg/normal																																																																																																		
SED	< 0.1 Kj/Kg																																																																																																		
B1+rms	119 uT																																																																																																		
Max B1+rms	119 uT																																																																																																		
PNS / level	22% / normal																																																																																																		
dB/dt	12.8 T/s																																																																																																		
Sound Pressure Level (dB)	-6.6																																																																																																		
PlanAlign	no																																																																																																		
REST slab	0																																																																																																		
Interactive positioning	no																																																																																																		
<b>Contrast</b>																																																																																																			
Scan type	Imaging																																																																																																		
Scan mode	3D																																																																																																		
technique	FFE																																																																																																		
Contrast enhancement	T1																																																																																																		
Acquisition mode	cartesian																																																																																																		
Fast Imaging mode	none																																																																																																		
Echoes	1																																																																																																		
Partial echo	no																																																																																																		
Shifted echo	no																																																																																																		
TE	user defined																																																																																																		
(ms)	5.6																																																																																																		
Flip angle (deg)	30																																																																																																		
TR	user defined																																																																																																		
(ms)	25																																																																																																		
Halfscan	no																																																																																																		
Water-fat shift	maximum																																																																																																		
Shim	default																																																																																																		
mDIXON	no																																																																																																		
Fat suppression	no																																																																																																		
Water suppression	no																																																																																																		
MTC	no																																																																																																		
Diffusion mode	no																																																																																																		
SAR mode	high																																																																																																		
B1 mode	default																																																																																																		
PNS mode	low																																																																																																		
Gradient mode	default																																																																																																		
SoftTone mode	no																																																																																																		
<p><b>Geometry</b></p> <table style="width: 100%; border-collapse: collapse;"> <tr><td>Uniformity</td><td>CLEAR</td></tr> <tr><td>FOV</td><td>FH (mm) 240</td></tr> <tr><td></td><td>AP (mm) 240</td></tr> <tr><td></td><td>RL (mm) 170</td></tr> <tr><td>Voxel size</td><td>FH (mm) 1</td></tr> <tr><td></td><td>AP (mm) 1</td></tr> <tr><td></td><td>RL (mm) 1</td></tr> <tr><td>Recon voxel size (mm)</td><td>0.94</td></tr> <tr><td>Fold-over suppression</td><td>no</td></tr> <tr><td>Slice oversampling</td><td>default</td></tr> <tr><td>Reconstruction matrix</td><td>256</td></tr> <tr><td>SENSE</td><td>yes</td></tr> <tr><td>    P reduction (AP)</td><td>2</td></tr> <tr><td>    S reduction (RL)</td><td>2</td></tr> <tr><td>k-t BLAST</td><td>no</td></tr> <tr><td>Overcontiguous slices</td><td>no</td></tr> <tr><td>Stacks</td><td>1</td></tr> <tr><td>    slices</td><td>170</td></tr> <tr><td>    slice orientation</td><td>sagittal</td></tr> <tr><td>    fold-over direction</td><td>AP</td></tr> <tr><td>    fat shit direction</td><td>F</td></tr> </table>	Uniformity	CLEAR	FOV	FH (mm) 240		AP (mm) 240		RL (mm) 170	Voxel size	FH (mm) 1		AP (mm) 1		RL (mm) 1	Recon voxel size (mm)	0.94	Fold-over suppression	no	Slice oversampling	default	Reconstruction matrix	256	SENSE	yes	P reduction (AP)	2	S reduction (RL)	2	k-t BLAST	no	Overcontiguous slices	no	Stacks	1	slices	170	slice orientation	sagittal	fold-over direction	AP	fat shit direction	F																																																									
Uniformity	CLEAR																																																																																																		
FOV	FH (mm) 240																																																																																																		
	AP (mm) 240																																																																																																		
	RL (mm) 170																																																																																																		
Voxel size	FH (mm) 1																																																																																																		
	AP (mm) 1																																																																																																		
	RL (mm) 1																																																																																																		
Recon voxel size (mm)	0.94																																																																																																		
Fold-over suppression	no																																																																																																		
Slice oversampling	default																																																																																																		
Reconstruction matrix	256																																																																																																		
SENSE	yes																																																																																																		
P reduction (AP)	2																																																																																																		
S reduction (RL)	2																																																																																																		
k-t BLAST	no																																																																																																		
Overcontiguous slices	no																																																																																																		
Stacks	1																																																																																																		
slices	170																																																																																																		
slice orientation	sagittal																																																																																																		
fold-over direction	AP																																																																																																		
fat shit direction	F																																																																																																		