

SIEMENS MAGNETOM TrioTim syngo MR B17

\\USER\MRRC Research\SiteProtocols\Pelphrey_Sequences\SAG T1 3D MPRAGE32ch_3nex

TA: 8:34 PAT: 2 Voxel size: 1.0x1.0x1.0 mm Rel. SNR: 1.00 SIEMENS: tfl

Properties

Prio Recon	Off
Before measurement	
After measurement	
Load to viewer	On
Inline movie	Off
Auto store images	On
Load to stamp segments	On
Load images to graphic segments	Off
Auto open inline display	Off
Start measurement without further preparation	On
Wait for user to start	Off
Start measurements	single

Routine

Slab group 1	
Slabs	1
Dist. factor	50 %
Position	L1.3 A31.1 F10.2
Orientation	Sagittal
Phase enc. dir.	A >> P
Rotation	0.00 deg
Phase oversampling	0 %
Slice oversampling	0.0 %
Slices per slab	176
FoV read	250 mm
FoV phase	100.0 %
Slice thickness	1.00 mm
TR	1230 ms
TE	1.73 ms
Averages	3
Concatenations	1
Filter	Prescan Normalize
Coil elements	HEA;HEP

Contrast

Magn. preparation	Non-sel. IR
T1	624 ms
Flip angle	9 deg
Fat suppr.	None
Water suppr.	None
Averaging mode	Short term
Reconstruction	Magnitude
Measurements	1
Multiple series	Each measurement

Resolution

Base resolution	256
Phase resolution	100 %
Slice resolution	70 %
Phase partial Fourier	Off
Slice partial Fourier	Off
Interpolation	Off
PAT mode	GRAPPA
Accel. factor PE	2
Ref. lines PE	24
Accel. factor 3D	1
Matrix Coil Mode	Auto (Triple)
Reference scan mode	Integrated
Image Filter	Off
Distortion Corr.	Off

Unfiltered images	Off
Prescan Normalize	On
Normalize	Off
B1 filter	Off
Raw filter	Off
Elliptical filter	Off

Geometry

Multi-slice mode	Sequential
Series	Ascending

System

Body	Off
HEP	On
HEA	On
SP4	Off
SP2	Off
SP8	Off
SP6	Off
SP3	Off
SP1	Off
SP7	Off
SP5	Off

Positioning mode	FIX
Table position	H
Table position	0 mm
MSMA	S - C - T
Sagittal	L >> R
Coronal	P >> A
Transversal	F >> H
Save uncombined	Off
Coil Combine Mode	Adaptive Combine
Auto Coil Select	Default

Shim mode	Tune up
Adjust with body coil	On
Confirm freq. adjustment	Off
Assume Silicone	Off
? Ref. amplitude 1H	0.000 V
Adjustment Tolerance	Auto
Adjust volume	
Position	Isocenter
Orientation	Transversal
Rotation	0.00 deg
R >> L	350 mm
A >> P	263 mm
F >> H	350 mm

Physio

1st Signal/Mode	None
Dark blood	Off
Resp. control	Off

Inline

Subtract	Off
Std-Dev-Sag	Off
Std-Dev-Cor	Off
Std-Dev-Tra	Off
Std-Dev-Time	Off
MIP-Sag	Off
MIP-Cor	Off
MIP-Tra	Off
MIP-Time	Off
Save original images	On

SIEMENS MAGNETOM TrioTim syngo MR B17

Sequence

Introduction	On
Dimension	3D
Elliptical scanning	Off
Asymmetric echo	Allowed
Bandwidth	320 Hz/Px
Flow comp.	No
Echo spacing	5.4 ms

RF pulse type	Normal
Gradient mode	Normal
Excitation	Non-sel.
RF spoiling	On

SIEMENS MAGNETOM TrioTim syngo MR B17

\\USER\MRRC Research\SiteProtocols\Pelphrey_Sequences\Sag_3D_short_ipat

TA: 3:25 PAT: 2 Voxel size: 1.0x1.0x1.0 mm Rel. SNR: 1.00 SIEMENS: tfl

Properties

Prio Recon	Off
Before measurement	
After measurement	
Load to viewer	On
Inline movie	Off
Auto store images	On
Load to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Start measurement without further preparation	On
Wait for user to start	Off
Start measurements	single

Routine

Slab group 1	
Slabs	1
Dist. factor	50 %
Position	R2.3 A48.7 F5.0
Orientation	Sagittal
Phase enc. dir.	A >> P
Rotation	0.00 deg
Phase oversampling	0 %
Slice oversampling	60.0 %
Slices per slab	160
FoV read	256 mm
FoV phase	100.0 %
Slice thickness	1.00 mm
TR	1900 ms
TE	2.96 ms
Averages	1
Concatenations	1
Filter	Prescan Normalize, Elliptical filter
Coil elements	HEA;HEP

Contrast

Magn. preparation	Non-sel. IR
TI	900 ms
Flip angle	9 deg
Fat suppr.	None
Water suppr.	None

Averaging mode	Long term
Reconstruction	Magnitude
Measurements	1
Multiple series	Each measurement

Resolution

Base resolution	256
Phase resolution	100 %
Slice resolution	50 %
Phase partial Fourier	6/8
Slice partial Fourier	6/8
Interpolation	Off

PAT mode	GRAPPA
Accel. factor PE	2
Ref. lines PE	24
Accel. factor 3D	1
Matrix Coil Mode	Auto (Triple)
Reference scan mode	Integrated

Image Filter	Off

Distortion Corr.	Off
Unfiltered images	Off
Prescan Normalize	On
Normalize	Off
B1 filter	Off
Raw filter	Off
Elliptical filter	On
Mode	Inplane

Geometry

Multi-slice mode	Single shot
Series	Interleaved

System

Body	Off
HEP	On
HEA	On

Positioning mode	REF
Table position	H
Table position	0 mm
MSMA	S - C - T
Sagittal	L >> R
Coronal	P >> A
Transversal	F >> H
Save uncombined	Off
Coil Combine Mode	Adaptive Combine
Auto Coil Select	Default

Shim mode	Tune up
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Silicone	Off
? Ref. amplitude 1H	0.000 V
Adjustment Tolerance	Auto
Adjust volume	
Position	Isocenter
Orientation	Transversal
Rotation	0.00 deg
R >> L	350 mm
A >> P	263 mm
F >> H	350 mm

Physio

1st Signal/Mode	None

Dark blood	Off

Resp. control	Off

Inline

Subtract	Off
Std-Dev-Sag	Off
Std-Dev-Cor	Off
Std-Dev-Tra	Off
Std-Dev-Time	Off
MIP-Sag	Off
MIP-Cor	Off
MIP-Tra	Off
MIP-Time	Off
Save original images	On

Sequence

Introduction	On
Dimension	3D
Elliptical scanning	Off
Asymmetric echo	Allowed
Bandwidth	160 Hz/Px

SIEMENS MAGNETOM TrioTim syngo MR B17

Flow comp.	No
Echo spacing	7.4 ms

RF pulse type	Fast
Gradient mode	Fast
Excitation	Slab-sel.
RF spoiling	On